

# WPAN (Wireless Personal Area Network) Setup Instructions


Many of the Rice Lake Medical Scales now have WPAN capability. Use the following instructions for the WPAN setup on these scales. WPAN technology is a wireless technology for connecting health and fitness devices because it's simple, secure and everywhere. WPAN allows the transfer of data from the scale wirelessly to WPAN-enabled devices such as cell phones, tablets, PDA or PCs.

### IMPORTANT


IMPORTANT: Due to the vast number of WPAN devices on the market and the large number of software packages available, there are many ways to pair and connect to the scale.


The steps used to pair the scale will vary, depending on the device/software being used. The scale is set up to be discoverable at all times.

1. Power on the indicator.
2. Ensure the WPAN device is on. The flag icon display on the indicator.
3. Press . The programs menu displays.
4. Type **add** into the search bar at the bottom.

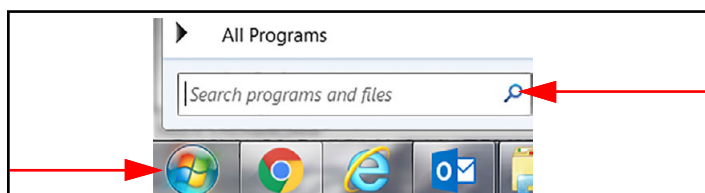



5. Under **Control Panel** select the option to add a WPAN device. The **Add a Device** window opens.

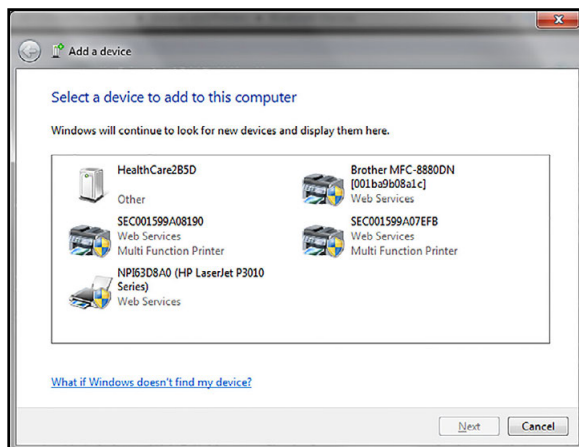


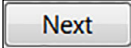
6. Select a WPAN device and press .
7. Select **Enter the device's pairing code**.

 **Enter the device's pairing code**  
The device comes with a pairing code.  
Check for one on the device or in the device manual.

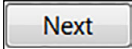8. Enter the device's pairing code (4829) in the text box. Check black circle displays next to the selected device.
9. Press . *This device has been successfully added to this computer displays.*

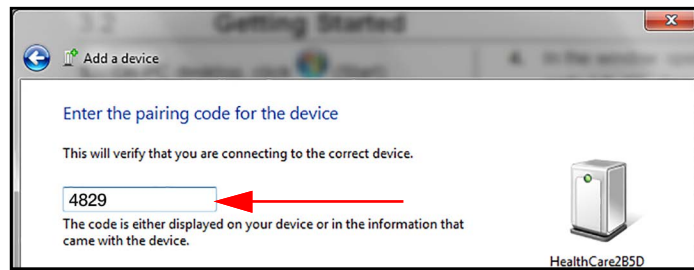

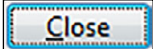10. Press .
11. Press the Show Hidden Icons arrow at the right side of the Task bar. 
12. Press the WPAN symbol to open.
13. In the Properties window, right-click on the device icon and select **Properties**.
14. Select the Hardware tab to check the correct Comm and then click **OK** to confirm that.

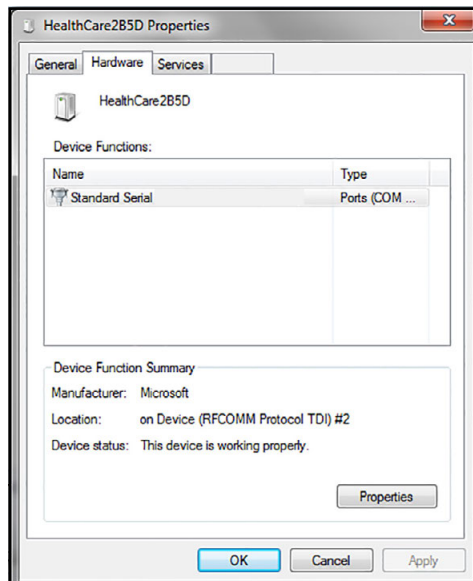

15. Choose any communication terminal in order to send commands through the WPAN communication. Once pairing is successfully completed, a bond will be formed between the two devices, enabling those two devices to connect to each other.



© Rice Lake Weighing Systems Content subject to change without notice.

230 W. Coleman St. • Rice Lake, WI 54868 • USA USA: 800-472-6703 • International: +1-715-234-9171